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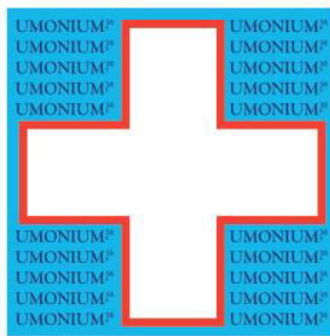
UMONIUM³⁸®

MEDICAL SPRAY

DISINFECTANT, CLEANER SPRAY FOR MEDICAL DEVICES

DISINFECTS AS SOON AS THE FIRST MINUTE

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MEDICAL DEVICE

Class IIb

BACTERICIDAL, MYCOBACTERICIDAL, FUNGICIDAL, VIRUCIDAL

Complies with: EN1040:2006, EN1275:2005, EN13727:2003, EN13624:2004, EN14348:2005, EN14476:2013, EN14561:2006, EN14562:2006, EN14563:2009

laboratoire HUCKERT'S INTERNATIONAL

STICKER BELAKANG BOTOL

EN INDICATIONS: Cleaning and disinfection for invasive medical devices before sterilisation. Cleaning and disinfection for non-invasive medical devices and their surfaces. Transport of soiled instruments. **INSTRUCTIONS:** **Transport:** Set the spray to the "foam" position and cover the devices with the foam. Make sure to cover it well to limit evaporation. **Cleaning/Pre-disinfection:** Set the spray to the "foam" position. Spray and scrub the devices to eliminate all dirt. Rinse thoroughly with water* or using a cloth dampened with water* if you are cleaning surfaces. **Disinfection:** Set the spray to the "fine spray" position. Spray over clean devices. Ensure a minimum contact time of 5 minutes. If you are cleaning invasive medical devices: Rinse with water*. If you are cleaning non-invasive medical device surfaces: Leave to dry or rinse with water* (or using a cloth dampened with water*) if necessary. **Clean drinking water at room temperature.* **PRECAUTIONS:** Handle in accordance with good industrial hygiene and safety procedures. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. Consult specific directions and material safety data sheet.

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1l e 35,2 oz.fl.

COMPOSITION UMONIUM³⁸ **MEDICAL SPRAY:** (AL) N-benzyl-N-dodecyl-N,N-dimethyl-ammonium chloride/N-benzyl-N,N-dimethyl-N-tetradecyl-ammonium chloride. 3,3 g/L.

LABORATOIRE HUCKERT'S INTERNATIONAL
20 Avenue Lavoisier
1300 Wavre - **BELGIUM**
Tel.: +32 (0)67 89 41 00
Fax: +32 (0)67 84 37 67
www.huckerts.net

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